



To Purchase, Visit Avobus.com or call [1-800-674-3655](tel:1-800-674-3655)

LIFEPAK[®] CR2 Defibrillator
with **LIFELINKcentral[™]** AED Program Manager

Operating Instructions

Important Information

Device Registration

Please register your device at www.physio-control.com. This will ensure that you are notified of any product updates.

Text Conventions

Throughout these operating instructions, special text characters (for example, **CAPITAL LETTERS** such as **LANGUAGE** button and **EVERYONE CLEAR**) are used to indicate labels and voice prompts.

LIFEPAK, LIFENET, and LUCAS are registered trademarks of Physio-Control, Inc. QUIK-STEP, LIFELINKcentral, cprCOACH, cprINSIGHT, Shock Advisory System, SAS, and ClearVoice are trademarks of Physio-Control, Inc. Specifications are subject to change without notice.

©2016 Physio-Control, Inc.
Publication Date: 07/2016

PN 3321515-024



To Purchase, Visit AvoBus.com or call [1-800-674-3655](tel:1-800-674-3655)

Contents

Introduction	9
About Automated External Defibrillators	11
Terminology	12
About the LIFEPAK CR2 Defibrillator	13
Safety Information	17
Terms	19
General Dangers and Warnings.....	19
Symbols	21
Chapter 3 Getting Started	25
Unpacking and Inspecting Your LIFEPAK CR2 Defibrillator	27
Where to Locate Your LIFEPAK CR2 Defibrillator	29
Controls, Indicators, and Labels.....	30
Chapter 4 Using the Defibrillator	33
Warnings and Cautions	35
Responding to a Sudden Cardiac Arrest Emergency	36
Additional Training Resources.....	40
Troubleshooting Tips	40
Chapter 5 LIFELINKcentral AED Program Manager	43
Overview	45
Features	45
Logging in to Your Account.....	46
Setting Up Wi-Fi.....	47
Initiating Wireless Connection	49
Connecting with Wi-Fi	50
Connecting with Cellular.....	51
Connecting with USB	52
Confirming AED Status Online.....	53
Updating Setup Options and Software	54
Troubleshooting Tips	58

Chapter 6 LIFENET System	59
Overview	61
Features	61
Logging in to Your Account.....	62
Setting Up Wi-Fi.....	63
Initiating Wireless Connection	65
Connecting with Wi-Fi	66
Connecting with Cellular.....	67
Connecting with USB	68
Confirming AED Status Online.....	69
Updating Setup Options and Software	70
Troubleshooting Tips	74
Chapter 7 Caring for the Defibrillator	75
Maintaining a State of Readiness	77
Replacing the Electrodes.....	79
Maintaining the Battery.....	81
Storing the Defibrillator.....	82
Cleaning the Defibrillator	82
Obtaining Authorized Service.....	83
Service Life	83
Recycling Information.....	83
Accessories, Supplies, and Training Tools.....	84
Warranty Information	84
Appendix A Specifications	85
Specifications	87
Appendix B Voice Prompts	91
Voice Prompts	93
Appendix C Defibrillator Setup Options	97
Setup Options.....	99

Appendix D Shock Advisory System and cprINSIGHT Analysis Technology	103
Overview	105
Shock Advisory System.....	105
cprINSIGHT Analysis Technology Performance.....	110
Appendix E Electromagnetic Compatibility Guidance	113
Electromagnetic Emissions.....	115
Canadian Standard ICES-003 Declaration	115
Electromagnetic Immunity	116
Separation Distances.....	118
Wireless Specifications.....	119
Cellular Specifications	119
Index	121

Introduction

This chapter provides a brief introduction to the LIFEPAK CR2 defibrillator and its features.

About Automated External Defibrillators.....	11
Terminology	12
About the LIFEPAK CR2 Defibrillator	13



To Purchase, Visit Avobus.com or call [1-800-674-3655](tel:1-800-674-3655)