

TIVA2 Washers

Thermal High Disinfector Line

Dental and medical practices need strict cleaning, disinfection, and sterilization of instruments before they can be used on patients. Thermal High Disinfection employs moist heat above 194° for a minimum of 5 minutes to kill bacteria and viruses in the washing phase. The TIVA2 line provides:

- Thermal High Disinfection to achieve a complete decontamination cycle.
- Advanced control system and an easy to use interface

TIVA2-H TD

Under Counter Washer



TIVA2 TD

Counter Top Washer



High end washers for the decontamination of complex instruments

TIVA2 Thermal High Disinfector washers provide a flexible turnkey solution for washing, disinfecting and drying diverse medical instruments for a complete decontamination cycle.



Pre-wash

Cold water removes soiled material



Detergent wash

Cleaning with warm demineralized water & suitable alkaline detergents



1st Rinse

Rinse with warm water to remove all cleaning agent residues, second rinse when required



Thermal disinfection

Water at a temperature above 194°F for a minimum of 5 minutes



Hot air drying

The dried instruments are ready for the next sterilization phase



To Purchase, Visit Avobus.com or call [1-800-674-3655](tel:1-800-674-3655)

TIVA2/TIVA2-H TD are fully-automatic thermal high disinfectors for all practices.

Features:

- Touch panel, color graphic display and real-time monitoring
- A total of 40 programs: Preset and programmable cycles
- Air drying prefilter + HEPA H14 filter
- Integrated water softener
- 2 dosing pumps with flowmeter for alkaline detergent and neutralizer (additional pump optional for TIVA2-H TD)
- USB/Printer for data documentation
- Provides complete & total evacuation of previous water (ETS)

Utility Requirements:

Water Connections

- Cold water connection for TIVA2 (default)
- Connection for demineralized water
- Integrated Water Softener with Salt Box
- R.O. water system is required
- Failure to incorporate an RO system with our TIVA unit will void your warranty

Electrical Connections

- 208+240V 1 phase 60Hz


Dimensions:

TIVA2-H TD Under Counter Washer

- (WxDxH) 23.4"x21.3"x32.2"
- Usable Chamber Volume: 65L

TIVA2 TD Counter Top Washer

- (WxDxH) 23.4"x21.3"x23.6"
- Usable Chamber Volume: 65L

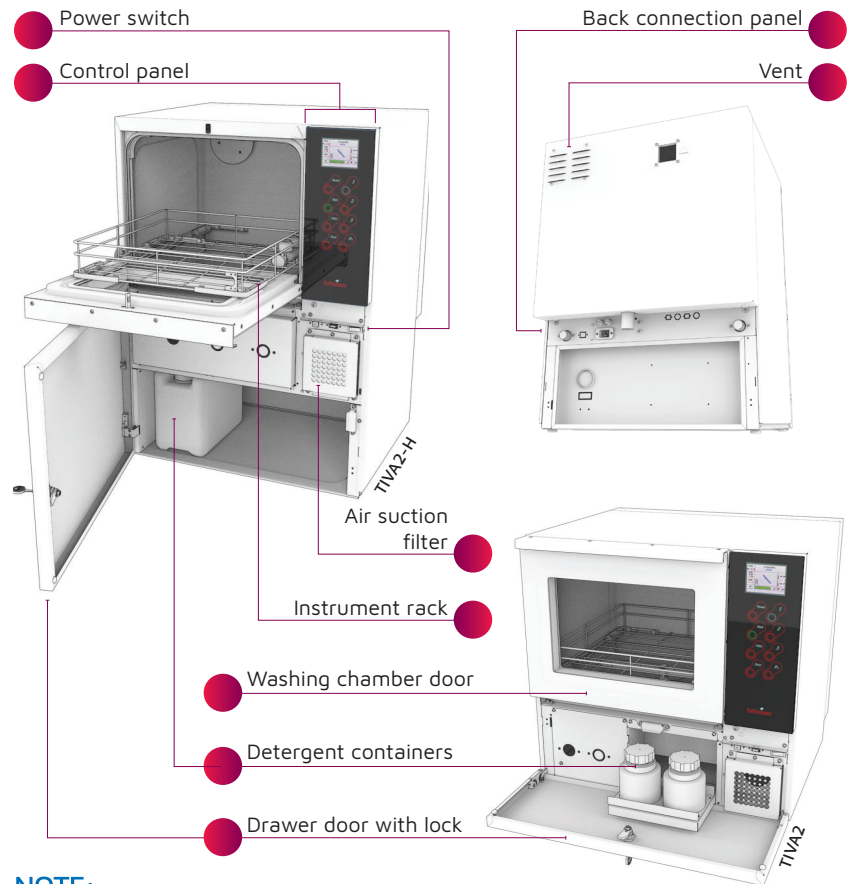

NOTE:

The front and back panel of the TIVA2 and TIVA2-H are identical



Sterisil® Ac+ System

Convenient, on demand autoclave water, made right in your practice



STERISIL23

T-Clean Tuttanauer Line of Detergents & Neutralizers

-User friendly for use in your Washer

T-Clean
TIVA
Detergent



T-Clean
TIVA
Neutralizer:



T-Clean
Prep
Foam

PF0006
500 ml
with sprayer



Indicators for Cleaning Efficiency Testing

Cleaning Indicator
(High Level)



Cleaning Indicator
Holder for Washer



Avobus

Tuttanauer USA Co. Ltd.

25 Power Drive, Hauppauge, NY 11788

Tel: +800 634 5836 / +631 737 4950, Fax: +631 737 0720

To Purchase, Visit Avobus.com or call [1-800-674-3655](tel:1-800-674-3655)

www.tuttanauerUSA.com

Learn from our experts:
Visit our blog: tuttanauerusa.com/blog

